



Empowering Aging with Dignity,  
Independence, and Innovation.

Daily Aids, So Easy.

## Bath Chair Installation Instructions SC6058E



**When you open the package, you will have the following accessories :**


Number	Picture	Name	Quantity
A		Backrest	1
B		Seat	1
C		Arm rest	2
D		Leg	4

**Assembly Instructions:**

- Take out the seat (Part# B), then you need to turn seat upside down, then put the four legs (Part# D) into the groove and rotate them to tighten.



- The leg (Part# D) have five levels of adjustment. Please adjust them to the same height.

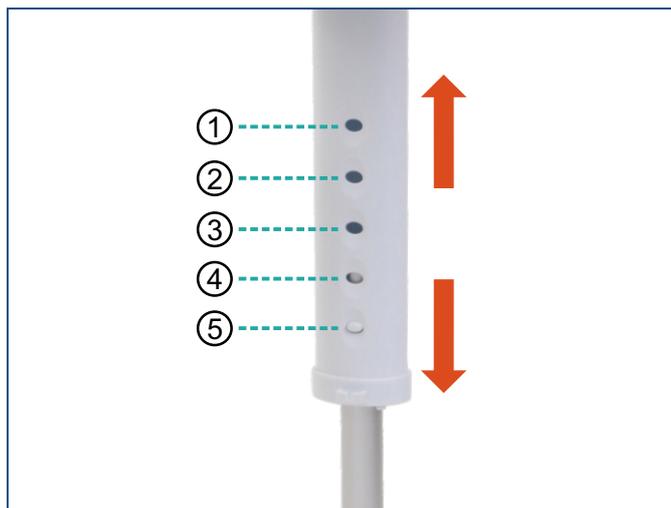




- Insert the arm rest (Part# C) into the grooves on both sides of the seat (Part# B).



- Finally, insert the backrest (Part# A) into the two rear slots of the seat (Part# B).



## Function Introduction:

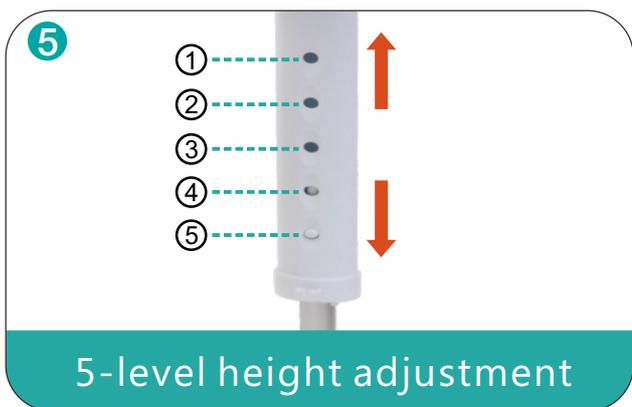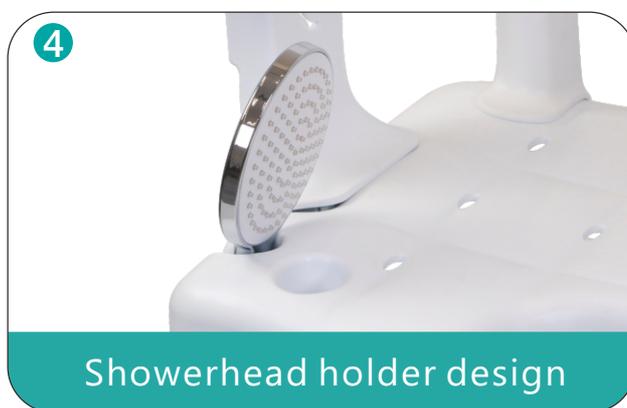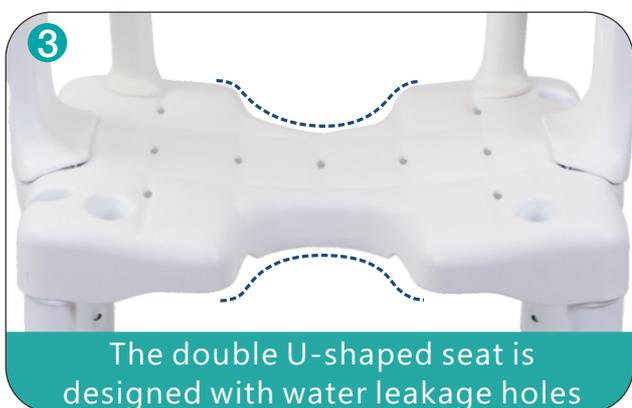

✉ [rd@suncare-medical.com](mailto:rd@suncare-medical.com)

🌐 <http://www.suncare-medical.com>

**About Us:** Soaids is a trusted mobility and care solution brand dedicated to enhancing the lives of seniors, including those with mobility issues. With thoughtful design, quality materials, and a deep understanding of aging needs, our innovative solutions empower older adults to live more independently and confidently, whether at home, outdoors, or in care facilities.

**At Soaids,** we believe that growing older should not mean giving up independence. We design every product with empathy, dignity, and purpose—to support seniors in aging gracefully, securely, and confidently.

With our in-house R&D, quality manufacturing, and deep empathy for the elderly, we create solutions beyond products—they are partners in daily life.

**Join Soaids in making life easier, safer, and more enjoyable.**